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Attention:
Customers who received one or more of the products listed below.

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All joints that have been manufactured and supplied since the sales start of the product 3R62* Pheon in calendar week 48/2014 are affected. The affected Serial Numbers are listed in the attached table.

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The Serial Number is located on the bottom side of the upper joint section, below the blue cover cap.

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It has shown that the locking function may fail under certain conditions.

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If the locking pin is not completely pulled out, edge loading in the locking surface causes a deformation of the locking pin.

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Due to the deformation the locking pin may get stuck in the guiding of the upper section hindering the lock from engaging automatically in extended position. The stability of the joint is then guaranteed only by the polycentric joint design.

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Dependent on the patient's physical status, the prosthetic alignment and the kind of loading, this may result in buckling of the loaded joint.

Indicazioni per risolvere il problema / Advise on action to be taken by the Customer:

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Please send the following information using the attached reply sheet attached until **July 31st, 2015**:

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- Catalog and Serial Number / quantity of joints still on your stock
- Catalog and Serial Number / quantity of joints patients have been fitted with and where the locking function is activated
- Catalog and Serial Number / quantity of joints patients have been fitted with and where the locking function is not activated

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Please return **all joints** with activated locking function patients have been fitted with as well as all **joints in stock** until **August 31st, 2015** to your local Otto Bock Service Center.

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All joints with non-activated locking function may be continuously used by the patients until further notice. As long as the locking function is not activated, there is no safety risk to the patient. At a later time, however, these joints also have to be replaced with re-designed Pheon joints. Detailed information on this will follow within the next months.

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Otto Bock Italia S.r.l. u.s.