Elettrocardiografi ECG MAC

2098702-015 Revisione A
29 Agosto 2016

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Bibliografia

1. Roffi, M., et al., *2015 ESC Guidelines for the management of acute coronary syndromes in patients presenting without persistent ST-segment elevation*. Task Force for the Management of Acute Coronary Syndromes in Patients Presenting without Persistent ST-Segment Elevation of the European Society of Cardiology (ESC), 2016. **37**(3): p. 267-315.
2. Priori, S.G., et al., *2015 ESC Guidelines for the management of patients with ventricular arrhythmias and the prevention of sudden cardiac death*. La Task Force for the Management of Patients with Ventricular Arrhythmias and the Prevention of Sudden Cardiac Death of the European Society of Cardiology (ESC) sottoscritta da: Association for European Paediatric and Congenital Cardiology (AEPC), 2015.
3. Kristensen, S.D., et al., *2014 ESC/ESA Guidelines on non-cardiac surgery: cardiovascular assessment and management*. La Joint Task Force on non-cardiac surgery: valutazione cardiovascolare e gestione della European Society of Cardiology (ESC) e della European Society of Anaesthesiology (ESA). Vol. 35. 2014. 2383-2431.
4. Rasmussen, M.B., et al., *Diagnostic performance and system delay using telemedicine for prehospital diagnosis in triaging and treatment of STEMI*. Heart, 2014. **100**(9): p. 711-715.
5. Tödt, T., et al., *Strategies To reduce time delays in patients with Acute coronary heart disease treated with primary PCI—the STOP WATCH study: a multistage action research project*. BMJ Open, 2013. **3**(9).
6. Peterson, E.D., et al., *A call to ACTION (acute coronary treatment and intervention outcomes network): a national effort to promote timely clinical feedback and support continuous quality improvement for acute myocardial infarction*. Circulation. Cardiovascular quality and outcomes, 2009. **2**(5): p. 491-9.
7. Lassen, J.F., H.E. Botker, and C.J. Terkelsen, *Timely and optimal treatment of patients with STEMI*. Nat Rev Cardiol, 2013. **10**(1): p. 41-48.
8. Danchin, N., et al., *Comparison of thrombolysis followed by broad use of percutaneous coronary intervention with primary percutaneous coronary intervention for ST-segment-elevation acute myocardial infarction: data from the french registry on acute ST-elevation myocardial infarction (FAST-MI)*. Circulation, 2008. **118**(3): p. 268-76.
9. Montalescot, G., et al., *STEMI and NSTEMI: are they so different? 1 year outcomes in acute myocardial infarction as defined by the ESC/ACC definition (the OPERA registry)*. Eur Heart J, 2007. **28**(12): p. 1409-17.
10. Gitt, A.K., et al., *BENCHMARKING IN THE MYOCARDIAL INFARCTION REGISTRY RHEINLAND-PFALZ (MIR-RLP) RESULTED IN INCREASE IN PRE-HOSPITAL ECG-RECORDING AND REDUCTION IN ISCHEMIC TIME TO REPERFUSION IN STEMI*. Journal of the American College of Cardiology, 2016. **67**(13_S): p. 658-658.
11. Hamm, C.W., et al., *ESC Guidelines for the management of acute coronary syndromes in patients presenting without persistent ST-segment elevation: The Task Force for the management of acute coronary syndromes (ACS) in patients*

presenting without persistent ST-segment elevation of the European Society of Cardiology (ESC). European Heart Journal, 2011. 32(23): p. 2999-3054.

12. *Norma internazionale 60601-2-25:2011 sui Dispositivi elettromedicali. Particular requirements for safety, including essential performance, of recording and analysing single channel and multichannel electrocardiographs, I.E.C. (CEI), Editor 2011, International Electrotechnical Commission (IEC). p. 190.*

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