Gentian AS
 Bjornasveien 5
 N-1596 Moss
 Norway
 T: +47 99 33 99 05

URGENT FIELD SAFETY NOTICE / DEVICE RECALL

Wednesday, 12 July 2017

Commercial name: Gentian Cystatin C Calibrator Kit
 Beckman Coulter part No.: A52763
 Kit lot numbers: 0006 and 0008

Dear Valued Customer,

Gentian AS has initiated a voluntary field action regarding two specific lots of Gentian Cystatin C Calibrator Kit (lot 0006 and 0008). Our records indicate that at least one of the affected lot numbers have been shipped to your account.

The identified lots of calibrators resulted in a high bias when used on a DxC instrument. Potential risks associated with the increased bias is an underestimation of the Cystatin C value, which again gives an overestimation of GFR. The bias should be detected when using controls for validation of the calibration curve. In worst case, this is not detected by the controls, and the bias may be close to 20 %.

Please note, an internal investigation has verified that the AU instrument platform is not affected by this issue. The impact to the IMAGE platform has not been assessed. However, no issues with the IMAGE platform have been reported to date. To minimize risk, regardless which platform you have, please discard any inventory of the affected lots (0006 and 0008).

What causes this calibrator problem?

The root cause is related to the antigen used in calibrator lots 0006 and 0008. For affected instruments, the calibration curve is influenced by the instrument and application settings and may give a high bias.

What measures has Gentian taken?

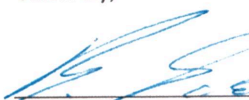
Actions have been taken towards the antigen supplier, which has resulted in improved quality of antigen raw material supplied to Gentian. Gentian has now included additional verification steps to ensure antigen quality prior to use.

What action is Gentian asking you to take?

- Please immediately stop using the affected calibrator lot 0006 and 0008
- Share this notification with other relevant personnel in your organization

We regret any inconvenience this may cause you and appreciate your patience. Gentian AS is committed to providing high quality products and ensuring customer satisfaction. If you have any questions, please do not hesitate to contact your local Beckman Coulter representative.

Sincerely,

 Bård Sundrehagen
 Chief Executive Officer

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