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Paola Canfora
Quality, Regulatory & Compliance Manager

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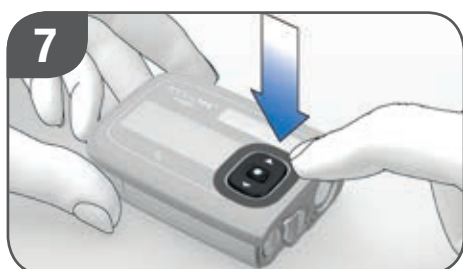
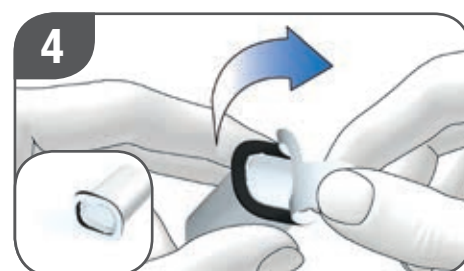
Key Frames

Instructions for use

A Cleaning tissue
B Applicator
C Applicator

The Accu-Chek Insight key frames consist of a cleaning tissue (A) and two applicators with black frames (B, C). White protective films cover the black frames. The applicators are necessary to place black frames around the keys of the insulin pump. This avoids accidental operation of the pump. Read and follow the instructions in the user's manual that came with your Accu-Chek Insight insulin pump carefully.

1

- Check if a protective film is stuck on the pump. If a protective film is stuck on, remove it.
- Stop the pump before mounting the frames. From the status screen, press . The main **Menu** appears.
- Press to move to **Stop pump**.
- Press to select. The **Pump stopped** screen appears. You can now disconnect the infusion set from your body and take off your pump.

2

- Hold the pump in an upright position. Turn the adapter with the tubing anti-clockwise.
- When you reach an angle of 120°, remove the adapter with the cartridge. The pump starts rewinding the piston rod. The battery remains in the battery compartment.

3

- Read the safety instructions on the packaging of the cleaning tissue (A).
- Wipe the front and the top of the pump thoroughly with the cleaning tissue (A). Make sure to wipe the area around the front and quick bolus keys.
- Let the keys dry.

4

- Remove the white protective film from the applicator (B).

5

- The applicator (B) has a symmetrical design. Hold the applicator (B) above the front keys of the pump.
- Place the applicator (B) on the front keys of the pump. If you feel the applicator (B) surrounding the front keys, the applicator (B) is positioned correctly.
- Use 2 fingers to press the black frame of the applicator (B) firmly onto the pump.

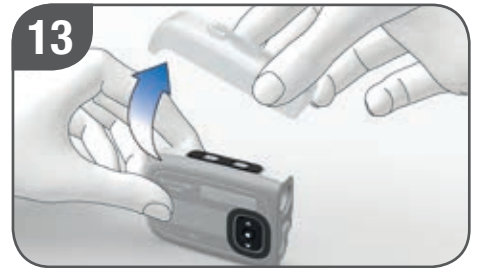
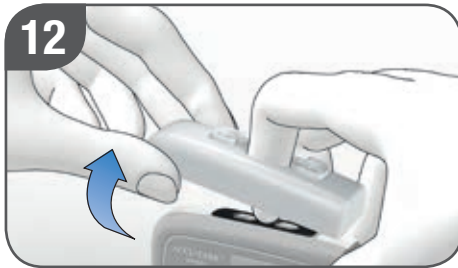
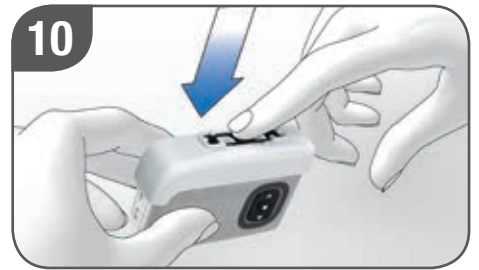
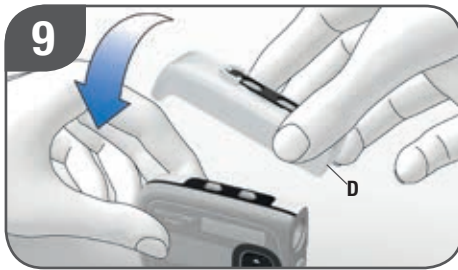
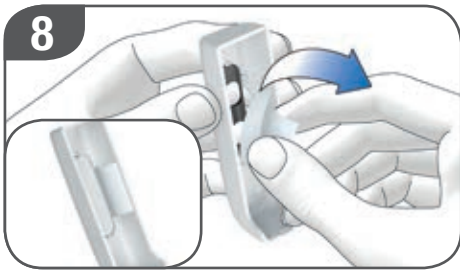
6

- Remove the applicator (B).

7

- Swipe with the finger over the black frame and press it firmly all the way around.

Turn the page!

- 8**
- Remove the white protective film from the applicator (C).
- 9**
- Place the pump on a flat surface. The quick bolus keys are pointing upwards.
 - Place the applicator (C) on the pump. The flat end (D) of the applicator is aligned with the flat end of the pump.
- 10**
- Use the finger to press the black frame in the middle of the applicator (C) firmly onto the pump.

- 11**
- Press the black frame firmly in the middle.
- 12**
- Keep pressing the black frame firmly while lifting the applicator (C). Your finger will be in the recess of the applicator (C).
- 13**
- Remove the applicator (C).
- 14**
- Swipe your finger over the black frame and press it firmly onto the pump.

- 15**
- You can now insert the cartridge and connect the adapter.
 - Dispose of the cleaning tissue (A), the applicators (B, C) with their protective films, and all packaging material according to the local regulations.

Only for:
Accu-Chek Insight insulin pump

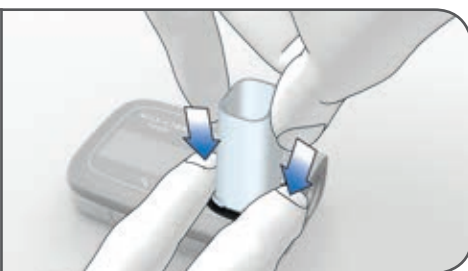
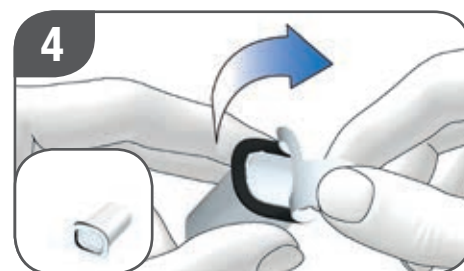
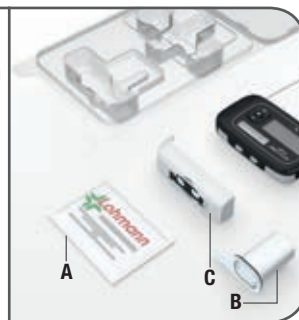
ACCU-CHEK and ACCU-CHEK INSIGHT are trademarks of Roche.
All other product names and trademarks are the property of their respective owners.

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© 2018 Roche Diabetes Care
Roche Diabetes Care Italy S.p.A.