

URGENT FIELD SAFETY NOTICE**Lifetech CeraFlex™ Atrial Septal Defect (ASD) Occluders****Models: See Appendix A**March 3rd, 2016**Dear Risk Manager or Health Care Professional:**

The purpose of this letter is to advise you that Lifetech is initiating an Urgent Field Safety Notice for a sub-set of Lifetech CeraFlex™ Atrial Septal Defect (ASD) Occluders listed in the Appendix A. This Field Safety Notice is being initiated due to the potential for the device to not properly take its intended final shape following unsheathing. Lifetech is the legal manufacturer of the CeraFlex™ ASD Occluder. According to Lifetech's records, you have received one or more of the potentially affected occluders.

Through February 2nd, 2016, Lifetech has received six (6) reports, from customers regarding CeraFlex™ ASD Occluders that do not properly take on their intended final shape. There have been no reports of any adverse patient impacts as a result of this issue - all involved devices were either retrieved before release, or were not used for the procedure. This issue is likely to be noticed prior to final release of the CeraFlex™ ASD Occluder and the device is designed to be fully retrievable. A potential hazard would be a delay of the procedure.

After review of the customer reports, Lifetech concluded that at least 5 out of 6 reported devices are related to over-sizing, i.e., over-sized occluders were used in the procedures that contributed to the malformation.

Although the reports received appear to be related to over-sizing, during the course of investigation we noticed a sub-set of CeraFlex™ Atrial Septal Defect (ASD) Occluders that appears more susceptible to malformation during the over-sizing issue.

For affected product that has been used, no action is necessary and patients should continue to be managed in accordance with your standard patient management protocol.

Customer Advice/Actions:

- 1.) Please review your inventory for CeraFlex™ ASD Occluders, identify the unused CeraFlex™ ASD Occluders in your inventory, with affected lots as listed in Appendix A.
- 2.) Return all identified CeraFlex™ ASD Occluders in your inventory to Lifetech by contacting Ms. Cherry Jing at +86 755 8602 6250 (ext.8807) or by email at cherry.jing@lifetechmed.com. Your Lifetech representative can assist you in the return of affected product as necessary.

Note that the Lifetech SteerEase™ Introducer, which is used together with the CeraFlex™ ASD Occluder, is not impacted by this Field Safety Notice.

For questions related to this notification, please contact your representative. If you require assistance in contacting your representative, please contact me.

The Competent Authority of your country has been notified of this issue.

Please share this notification with others in your organization as appropriate. If product within the scope of this notification has been forwarded to another facility, please alert the facility of this notification. Lifetech is committed to ensuring the product it distributes meets the highest quality standards. We appreciate your cooperation and apologize for any inconvenience this issue may cause.

Sincerely,

Lily Shi
RA Director



Mar. 3, 2016

Lifetech Scientific (Shenzhen) Co., Ltd.

Tel: +86 755 8602 6416

Email: shixiaoli@lifetechmed.com.

Appendix A:

Table A.1 The Affected Products and Lot Numbers

Important Notes:

- For those territories under Medtronic Distributionship, Lot number is the first six (6) digital number of series number (SN);
- For other territories, a lot number is specified in the label.

Product Code	Lot #	Product Code	Lot #
LT-ASDf-12	140904	LT-ASDf-20	141015
LT-ASDf-12	140904	LT-ASDf-26	141015
LT-ASDf-06	140916	LT-ASDf-22	141018
LT-ASDf-08	140916	LT-ASDf-28	141018
LT-ASDf-10	140916	LT-ASDf-30	141018
LT-ASDf-12	140916	LT-ASDf-32	141018
LT-ASDf-12	140916	LT-ASDf-20	141024
LT-ASDf-14	140916	LT-ASDf-18	141027
LT-ASDf-16	140916	LT-ASDf-12	140916
LT-ASDf-18	140916	LT-ASDf-16	140916
LT-ASDf-20	140917	LT-ASDf-18	140916
LT-ASDf-22	140917	LT-ASDf-20	140917
LT-ASDf-24	140917	LT-ASDf-22	140917
LT-ASDf-26	140917	LT-ASDf-18	140918
LT-ASDf-18	140918	LT-ASDf-28	140918
LT-ASDf-28	140918	LT-ASDf-32	140921
LT-ASDf-30	140921	LT-ASDf-08	141014
LT-ASDf-32	140921	LT-ASDf-14	141014
LT-ASDf-16	141010	LT-ASDf-20	141014
LT-ASDf-08	141014	LT-ASDf-24	141014
LT-ASDf-14	141014	LT-ASDf-06	141015
LT-ASDf-20	141014	LT-ASDf-16	141015
LT-ASDf-24	141014	LT-ASDf-18	141015
LT-ASDf-06	141015	LT-ASDf-24	141015
LT-ASDf-10	141015	LT-ASDf-22	141018
LT-ASDf-12	141015	LT-ASDf-30	141018
LT-ASDf-16	141015	LT-ASDf-32	141018
LT-ASDf-18	141015		