Medimar S.r.l.
strumenti medicali

Medimar S.r.l. Unipersonale
Sede Legale: Via Ebro, 8 -20141 Milano
Tel. +39 02 57410545 Fax +39 02 57410566
E-mail: info@medimaritalia.it
PEC: medimarsrl@pec.ultraposta.net

Milano, 19/12/2017

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All lot numbers of ballons END BALL, reference: END T110

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A user informed us that the gastric wall of the patient had been damaged, during the inflation of the balloon

Actions destinées aux clients (utilisateurs) / Advise on action to be taken by the user:

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We remind you that it is imperative to follow the following recommendations when introducing and inflating the balloon:

- *The patient must be positioned in the left lateral position, the head in the axis of the spine, maintained by a pillow*
- *Passage through esophagus with the balloon continuously under endoscopic view.*
- *Position the balloon in the stomach, the mesial part of the introducer at least 5 cm from the cardia. The introducer must have sufficient space to allow the deployment of the balloon without compressing the gastric wall. The introducer must be visible in its entirety before beginning the inflation.*
- *If the introducer is not visible in its entirety, reposition the introducer correctly.*

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LABORATOIRE
endalis

Potential potential risk:

If inflation of balloon occurs without following these recommendations, there is a significant risk of damage to the gastric wall.

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The complications mentioned in the instructions for use will be modified.

The procedure for insertion the device of the instruction manual will be modified.

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This notice needs to be passed on all those who need to be aware within your organisation or to any organisation where the potentially affected devices have been transferred. (If appropriate)

Please transfer this notice to other organisations on which this action has an impact. (If appropriate)

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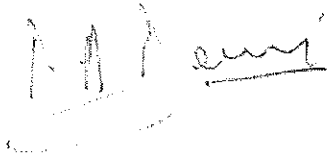
ENDALIS confirms that this Field Safety Notice has been approved by ANSM, French Competent Authority.

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